


# HOSE RETRACTOR

---

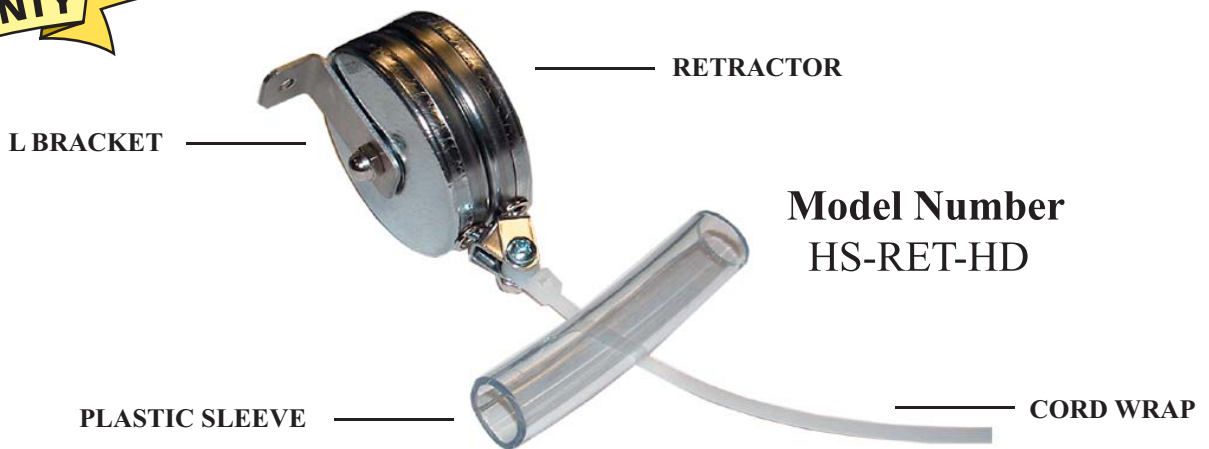

## Hose Retractor

Amvex Corporation offers a heavy duty hose retractor manufactured with the highest quality materials. Our hose retractor can be attached to a ceiling outlet or directly to the ceiling itself, please see installation instructions below. The retractor wire is manufactured out of 48 inches of heavy duty pvc coated cable.

## Installation Instructions

1. Remove any screw from the face.
2. Put the "L" Bracket where the screw from the outlet was removed.
3. Place the screw through the hole in "L" Bracket into the outlet.
4. Fasten screw with a screw driver so "L" Bracket is secure.
5. Attach the hose to the outlet.
5. Attach the hose to the outlet.
6. Place plastic sleeve on desired location of hose where you would like the cord wrap affixed.
7. Wrap the cordwrap around the hose and plastic sleeve (the sleeve provides a protection to the hose).
8. Pull the cordwrap snug to the hose, be careful not to over tighten as you may restrict the flow.