


LDRP MOUNTING CONCEPT

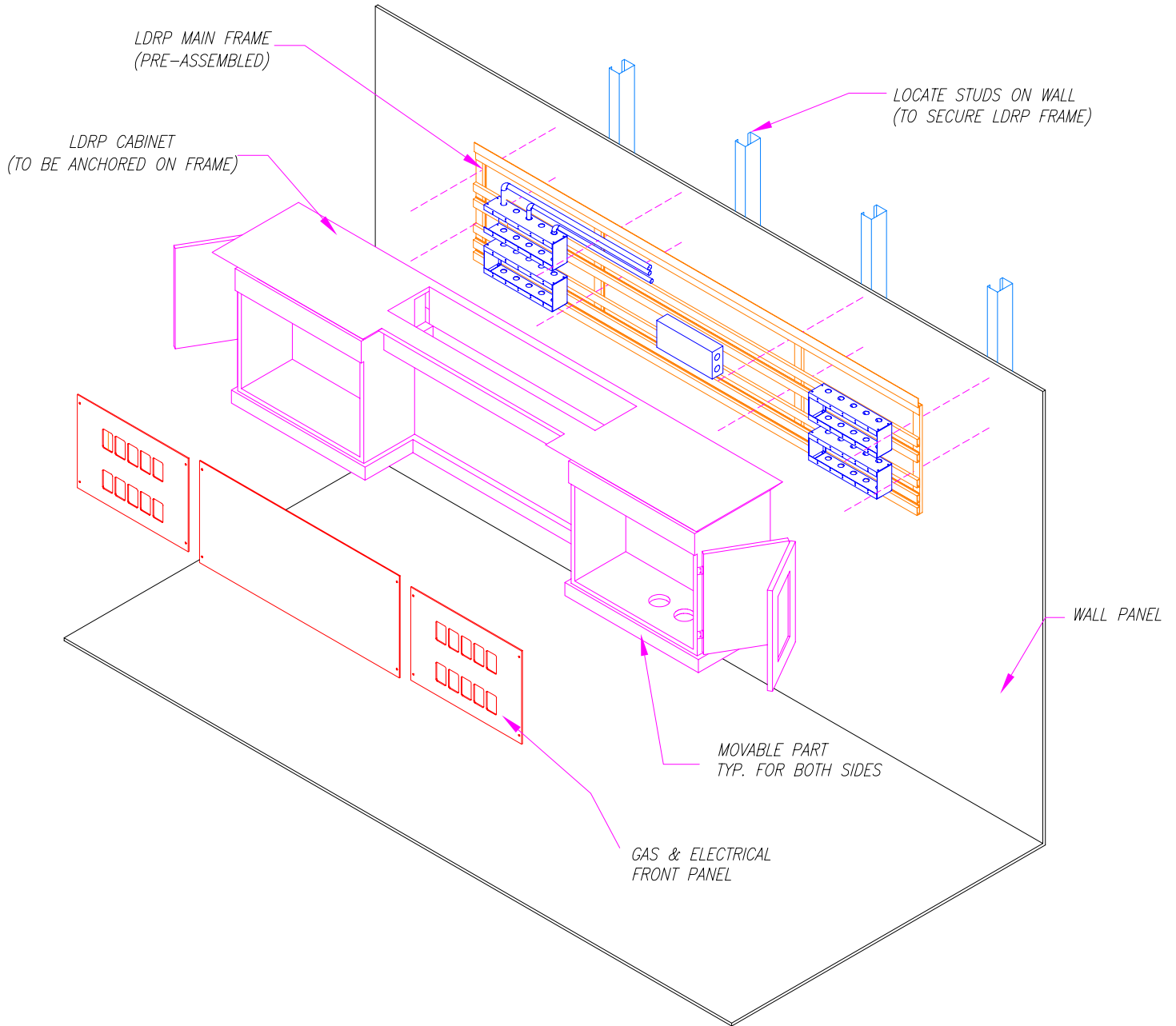

LDR Installations instructions

IMPORTANT: DO NOT SIT THE CABINET ON THE MOVABLE BOTTOM TRAY. ALWAYS LAY CABINET ON ITS BACK!!!!!!

1. Secure the frame to the wall.
2. Remove electrical access panel (middle).
3. Hang up the cabinet over the frame. Center the cabinet on the frame. Cabinet openings from left and right sides should match with boxes from the frame.
4. Do all necessary connections for the light and bottom micro switches (see attached diagram).
5. Have left and right front panels for your unit handy. Pull all electrical devices out of the boxes. Put the front panel on the cabinet and match the cut-outs with the devices positions. Do this step for both sides of the cabinet. Adjust cabinet position for a better alignment if necessary.
6. Holding the front panel on the correct position, surface mount the devices in place and then secure the cover to the cabinet. (Drill pilot holes and attach with screws provided)
7. Install devices cover plates.
8. Anchor the cabinet to the frame using middle brackets attached to the cabinet.
9. Install the middle panel in place.
10. Insert the trim caps where necessary.
11. Install blood pressure device.
12. Install vacuum slides to the bottom using screws provided.

