





Chemetron Digital Alarm Retro-Fit Manual

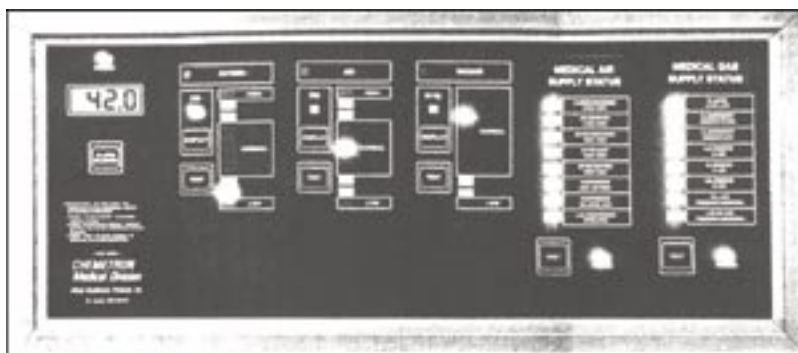


The Amico retrofit alarm will allow an existing Chemetron / Allied to be replace, without removing the existing back box and pipe-stubs.


Remove all internal components of the Chemetron alarm (mother board, power supply, sensors and all wiring). Remove the front frame and the display modules from back box.

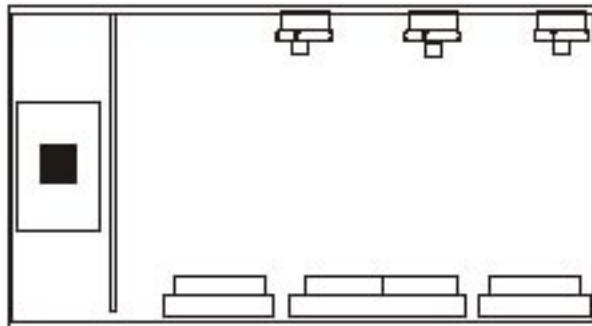

Ensure that the back box is clean of ALL screws, studs, stand-offs and insert tabs. Bend the lower front lip of the alarm box. Ensure that the lip is flat.

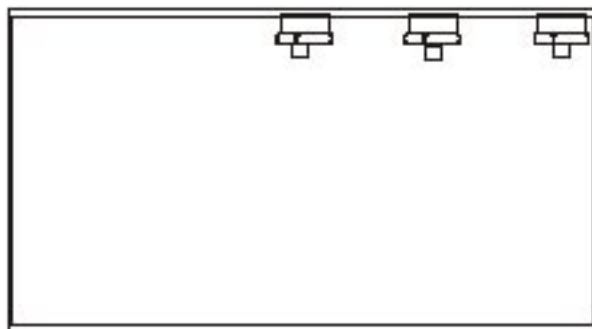


Add the power supply to the far left corner of the back box assembly. Use 4 tapping screws to hold power supply into box. Use tapping screw for grounding wire.

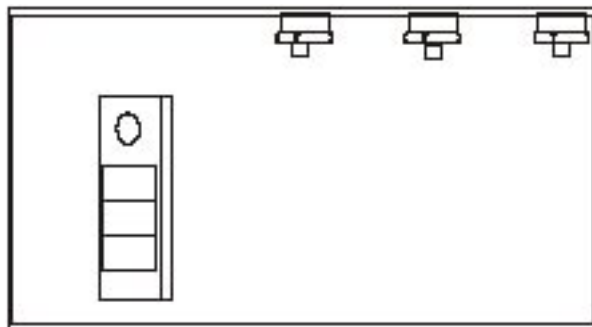


Remove the brass hex fitting from the upper portion of the box pipe-stub assembly. Replace with the brass hex and female quick-connect fitting supplied by Amico.

Connect into place the Alert-2 sensor, which contains the male quick-connect fitting, instead of a DISS fitting (nut and nipple).

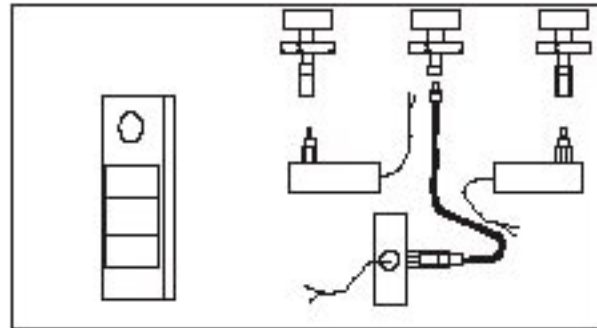

Add Amico filler frame to the wall surface. Ensure that the box on the wall is in the center of the filler frame.

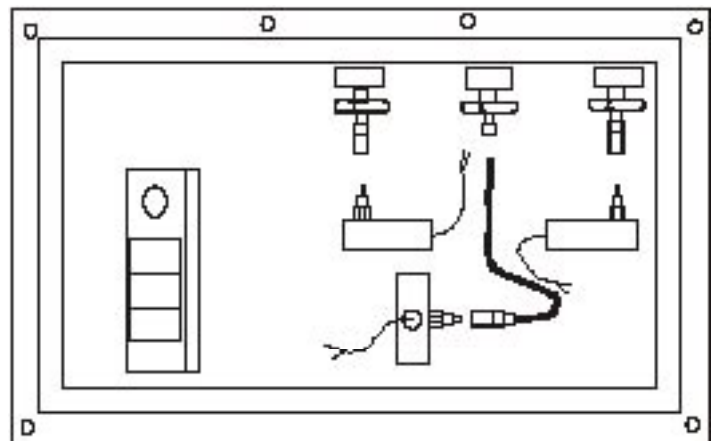

Install front assembly and follow the “Installation & Operation Manual of Alert-2 Alarm” (Also supplied with retro-fit alarm).





14 Madison Rd., Fairfield, NJ, 07004, USA
85 Fulton Way, Richmond Hill, ON, L4B 2N4, Canada

Phone: 1-877-GO AMICO (1-877-462-6426)
Fax: (905) 764-0862 www.amico.com